

**Dux Dental**  
 Zonnebaan 14, NL-3542 EC Utrecht  
 The Netherlands  
 Tel. +(31) 30 241 0924  
 Fax. +(31) 30 241 0054  
 Email: info@dux-dental.com  
 www.duxdental.com

Trust.Worthy.Innovation.



## The perfect mix with the Alginator

The Alginator mixing machine makes a smooth, bubble-free mix in a jiffy, while eliminating waste and reducing mixing and clean-up time significantly.

### Easy handling

The alginate mixing machine is mainly characterized by its low weight (< 1,5 kg).

It is easy for right-handers and left-handers to operate the mixer due to its ergonomic and slim design.

There is only one single mixing speed for a consistent and bubble-free mixing results every time.



## Is it a silicone? No, it's Image™ Alginate!



Image Alginate is dust free, very smooth, creamy and easy to mix.

The green color makes all details of your impression easily visible. Image Alginate has a fresh mint flavor and is a natural, 100% bio-degradable product.

*Image Alginate.  
 Unsurpassed smoothness  
 and amazing detail.*

		Fast set (2 Minutes)	Regular set (3 Minutes)
Image Alginate canister	500 g + canister	Ref. 27416	Ref. 27426
Image Alginate Value pack	10x 500 g + canister	Ref. 27417	Ref. 27427
Image Alginate Refill pack	11x 500 g	Ref. 27419	Ref. 27429
Image Alginate Institutional pack	30x 500 g	Ref. 27460	Ref. 27461
Image Alginate Singles	52 units/box	Ref. 27412	
Image Alginator	Mixing machine incl. 0,5 kg Alginate	Ref. 25241	Ref. 25232
Image Syringable impression-taking system	48 cartriloids, 1,8 ml (Syringe material blue)	Ref. 21600	
Image Syringable impression-taking system	Starter Kit	Ref. 21601	
Image Syringable impression-taking system	Starter Kit incl. DryProcessor	Ref. 25300	



## Image Alginate product features and benefits

### Water-loving formula

Uniform particle size assures a precise rate of absorption. All ingredients are soluble at the same rate yielding a quick, uniform and smooth mix.

### Unsurpassed surface smoothness and green color

Your impressions will have a glass-like finish and the green color makes them easy to read. You will see the difference in your stone models and in your provisional restorations.

### Dust-free

The alginate will not dust during dosing and mixing.

### Friendly to the environment

100% bio-degradable.

### Fresh mint flavor

Flavour for everyone.

## Image Syringable

Image Syringable is a cost saving, quick and precise two component alginate-hydrocolloid impression-taking system, ideal for single crowns, three unit bridges, removable partial dentures, inlays and onlays.



**2 Application.** Dispense Image Syringable rapidly and in bulk directly onto the marginal areas and contours of the moistened prepared teeth.




**4 Tray removal.** Hold until the alginate is completely set and remove the tray with a quick snap.



**3 Impression-taking.** Seat the tray with Image Alginate over the fluid syringe material.



**5 Rinse.** Rinse the impression for about 30 seconds under running water.

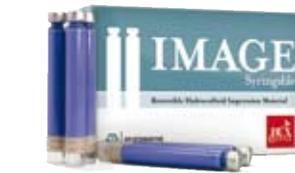

**1 Preparation.** Image Syringable is boiled and kept in the DryProcessor and after processing ready for use during the whole week.



**6 Disinfect.** Disinfect the impression using your preferred disinfectant, following the manufacturer's instructions.



**7 Pour.** Rinse the impression with tap water once more and pour immediately for best accuracy.

## Image Alginate Singles

The convenient, efficient and hygienic alginate solution for *high quality* alginate impressions. Image Alginate is now also available in single impression units: "Image Alginate Singles".

### Quality

Always the right measure of alginate and water. The mixture viscosity will be exactly the same every time.

### Efficient

No need to use a scoop and vial to measure alginate and water and no more space consuming alginate canister or bag.

### Hygienic

No more unwashed water vial gathering bacteria. A personal alginate packaging for every patient.



actual size

